



Medical  
Solutions, Inc.®



## ThermaCor® 1200 Drape For Use with the ThermaCor® 1200 Rapid Infuser



**WARM TO THE CORE™**



# ThermaCor® 1200 Drape

## INSTALLING THE THERMACOR® 1200 Drape

1. Secure the ThermaCor® 1200 to the IV pole. Verify that all of the surfaces of the ThermaCor® 1200 are clean and dry. If not, clean and dry surfaces before continuing.
2. Remove the ThermaCor® 1200 Drape from the packaging by tearing at the notch located on either the left side or right side of the pouch.
3. Unfold the ThermaCor® 1200 Drape.
4. With the absorbent side facing up (or shiny side down), position the Drape onto the ThermaCor® 1200 by separating the flaps of the slit, and enclosing the IV pole in the circular cut-out of the Drape.
5. Orient the drape as indicated on previous page. Ensure that the folded creases of the Drape are located close to the edges of the ThermaCor® 1200. The Drape is not intended to block the display of the ThermaCor® 1200.
6. Locate the adhesive between the Drape flaps - near the IV pole. Remove and properly discard the liner of the tape to expose the adhesive.
7. Overlap the flaps so that the Drape is securely wrapped around the IV pole. There should be minimal gaps between the IV pole and the Drape.
8. Locate the tape near the underside of the Drape on the backside of the Unit. Remove and properly discard the liner from the tape to expose the adhesive.
9. Place the adhesive of the Drape onto the backside of the ThermaCor® 1200 - ensuring that the creases of the Drape line up to the edges of the ThermaCor® 1200.
10. Load a Cassette. Check to be certain the cassette is in the locked position. Place the Drape over the top of the Cassette.

**Note:** The Drape should not be between the Cassette and the Pump.

## WARNINGS

The ThermaCor® 1200 Drape should not completely cover the air vent at the rear of the ThermaCor® 1200. Blocking the air vent may damage the system.

Do not alter or change any part of the ThermaCor® 1200 Drape.

Store at room temperature.

**3901 Centerview Drive  
Suite L  
Chantilly, VA 20151  
1-800-435-7600 (Phone)  
1-703-834-0039 (Fax)  
[www.msi-healthcare.com](http://www.msi-healthcare.com)**